SAREMCO Dental restorative system with apt



apt unibond



etch





(optional) etch

thoroughly

dry with oilfree air, keep dry







cure

Exposure time

light power	≥ 500 mW/cm²	≥ 1′200 mW/cm²
exposure time	20 sec.	10 sec.

bond

vlaga and massage

with oilfree. air, keep dry

Note: apt unibond is a self-etching-adhesive and is applicable with all common etch techniques (non-etch, selective etch, total etch, etc.).

drv



3





apt composite



line and cure (apt flow optional)



restore and cure layer by layer



restore and cure (small cavities) apt flow

restore

Exposure time for increments of 2mm

light power	≥ 500 mW/cm ²	≥ 800 mW/cm ²
apt all colours	40 sec.	20 sec.